




THE PROTOCOL OF USE

1: Placing chilled water and ice in the water tanks.

The pelletier option is available, on the console, in order to return the perfectly autonomous device.

2: Start the software with selection of the lens temperature and setting of the processing time.

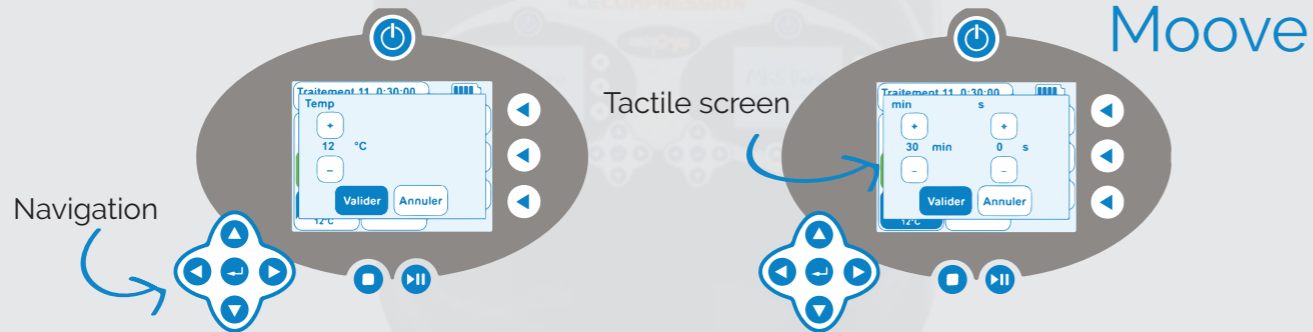

3: Filling the mask with chilled water, in order to reach the stated temperature objective.

3 sensors allow the automatic regulation of the temperature:

- Water tray temperature
- Lens temperature
- Temperature of the return flow in the mask

4: Placing the mask on the face in the room to wake up.



TECHNICAL CHARACTERISTICS

- Mask for single use or multiple use mask for cost optimization
- Vertical orientation of the chilled water circuit allowing to follow the path of the lymphatic circuit.
- Operates on mains or on battery up to 5 hours autonomies
- Water supply connectivity with non-return valve.
- Double navigation, touch screen and navigation arrow
- Application temperature up to 10 ° C allowing a reduction in treatment time.



First

EFFECTS

- **RELIEVES** pain
- **IMPROVES** patient comfort by postoperative
- **ALLOWS** analgesic action and anti-inflammatory
- **DECREASES** postoperative edema



Water tank

Traitement 1/1		⚡
0:30:00	Quitter	
EAU Manuel	Attelle genou	Air Statique
10°C/8°C		
8°C	⏸	▶
		75 mmHg

Setting up mask in the wake up room

Back in bedroom

Care from 24 to 72 h between 12° to 14°C

Patient exit

THE DEVICES OF EASYCRYO



Moove

One-way portable system allows 1 treatment on 1 patient.
The Moove is available on his console



Twin

System with two outputs allow to carry out 2 treatments on 2 patients.
The Duo is available on his console



First

One-way system allows for 1 treatment on 1 patient.
The First is available on his console

MAXILLOFACIAL SURGERY



TECHNICAL DETAILS

	TWIN	MOOVE / FIRST
Number of exits	2	1
min / max compression	0 mmHg / 75 mmHg	0 mmHg / 75 mmHg
Compression	dynamic/constant	dynamic/constant
Water temperature	10° à 15°C	10° à 15°C
Traitement eau chaude	Yes (45°C max)	No
Cycle of Cold and Rest	Fonction repeat	Fonction repeat
CE med	LNE/GMED 0459	



Zone ECOPARC
27 Rue Mezière Christin
34690 FABRÈGUES - France



www.easycryo.fr

Tél. +33 (0)4 99 64 21 05
Fax. +33 (0)4 99 64 21 06
olivia.beldame@easycryo.fr

